

Aerodesin[®] 2000

Ready-for-use alcohol based spray disinfectant
for the rapid disinfection of medical devices

Ready-for-use
For professional use

Composition

100 g solution contains :
32.5 g Propan-1-ol, 18.0 g Ethanol (denatured), 0.1 g Glutaral

Range of application

Rapid disinfection of

- Small surfaces and objects
- Surfaces difficult to access

Mode of application / dosage

Use undiluted only!

Spray Aerodesin[®] 2000 on objects and surfaces from a distance of about 30 cm. Objects and surfaces must be completely wetted (max. 50 ml/ m² of the floor area).

Disinfection by wiping according to the VAH procedure 2002, mechanical effect, dirty conditions		
bactericidal, levurocidal	100%	60 s
Tests according to the directive of DVV/RKI		
virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 1/2004	100%	30 s
adeno virus	100%	30 s
papova virus/polyoma virus/SV 40	100%	60 s
polio virus	100%	60 mn
EN-Norms (dirty conditions)		
EN 13624, fungicidal	100%	5 mn s
EN 14476 (MNV)	100%	30 s
virucidal	100%	60 mn

Warnings / storage

- Aerodesin[®] 2000 is flammable.
- Keep away from hot surfaces, sparks, open flames and/or any other incandescent objects!
Do not smoke.
- Keep disinfectant in a dry, cool but frost-free and well ventilated area, away from ignition sources and out of reach of children.
- Do not use beyond expiry date.

Properties

- Control of bacteria (incl. tb), fungi and viruses
- Rapid action and rapid drying after spraying on surfaces and objects
- Freshly scented
- Good wetting power
- Spray-disinfectant ready for use in spray bottles without aerosols

Special precautions

a) Counter-measures in case of improper use:

Aerodesin[®] 2000 is irritating by contact with the mucous membranes (eyes, throat, stomach, bowels etc.). In case of such irritations, dilute product immediately. If it has been swallowed, make patient drink plenty of water; after contact with other mucous membranes, rinse thoroughly with plenty of water. In all cases get medical attention immediately.

b) Miscellaneous:

Do not spray on materials that are not compatible with alcohol (coated, pickled surfaces, Plexiglas etc.).

c) Use Aerodesin[®] 2000 safely. Always read the label and product information before use.

Certified and mentioned in the relevant lists.

Standard packaging / transport units:

12 x 1 l bottle with sprayer
2 x 5 l canister

9736
9725

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219
www.lysoform.de, e-mail: kontakt@lysoform.de

1218-16