

Instructions for use - Please read carefully!

## Blanisol<sup>®</sup>-Pur

### Concentrate

**Highly concentrated special detergent for medical instruments  
(including endoscopes)  
Cleaning booster for cationic disinfectants  
Economical, gentle surface cleaner**

### For professional use

### COMPOSITION

100 g solution contains:

15-30g Non-ionic Surfactants = Isotridecanol ethoxylated, 1g alkyl dimethyl benzyl ammonium chloride, Fragrance

### RANGE OF APPLICATION

- Detergent for instruments and surfaces
- Cleaning booster for cationic disinfectants

### MODE OF APPLICATION / DOSAGE

Dilute before using!

**Blanisol<sup>®</sup>-Pur** is a liquid concentrate with which any use-dilution required could be prepared by adding water. To avoid foaming always fill up container with water first, then add **Blanisol<sup>®</sup>-Pur**.

- |   |              |
|---|--------------|
| • Cleaning of medical instruments                       | 0.25% - 1.0% |
| • Cleaning of water resistant surfaces                  | 0.25% - 0.5% |
| • Boosting of cleaning effect of cationic disinfectants | 0.25% - 0.5% |

### WARNINGS / STORAGE

- Keep detergents in a cool but frost free, well ventilated and dry place and out of reach of children.
- Wear adequate protective gloves, suitable protective clothing, and eye protection when handling the concentrated product.
- Do not use beyond expiry date.

### PROPERTIES

**Blanisol<sup>®</sup>-Pur** is a highly concentrated detergent for the gentle elimination of slight and heavy organic soil especially of blood, albumin, lipids.

We guarantee the chemical/microbiological compatibility with the following disinfectants: **Almyrol<sup>®</sup>**, **Lysoformin<sup>®</sup> spezial**, **Lysoformin<sup>®</sup> 3000** and **Lysoformin<sup>®</sup> rapid**.

### STANDARD PACKAGING / TRANSPORT UNITS:

12 x 1000 ml bottles

article No: 2113

### Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D-12247 Berlin

Tel.: +49-30-77992-0    Telefax: +49-30-77992-219

[www.lysoform.de](http://www.lysoform.de), e-mail: [kontakt@lysoform.de](mailto:kontakt@lysoform.de)

### Lysoform Schweizerische Gesellschaft für Antisepsie AG

Postfach 444, CH-5201 Brugg/Windisch

Telefon +41-56-441 69 81, Telefax +41-56-442 41 14

[www.lysoform.ch](http://www.lysoform.ch)

2020/04/06\_12