

Hospisept® Tuch (wipes)

Disinfecting and cleaning wipes for
surfaces, medical equipment and medical devices

Product information

Please read carefully!

Composition

1 moist **wipe** contains on average:

1.1 g Propan-1-ol, 0.32 g Ethanol 96 %, Butan-2-on, Dibutyl Adipate, Purified Water

Range of application

- Disinfection and cleaning of small surfaces, disinfection of medical devices, for example of contact glasses for diagnostic purposes, of dental equipment etc., and antimicrobial pre-cleaning of endoscopes
- To be used where it is not possible to spray the product or when rapid drying is important.
- Hygienic hand disinfection

Mode of application / dosage

- Wipe the surface (always make sure that it is completely wetted with the disinfectant!) and allow product to work until dry (about 5 minutes).
- Hygienic hand disinfection: 30 seconds. Wet hands completely with product. (use 2 wipes if necessary).
- Virucidal against enveloped viruses, incl. HBV/HCV/HIV and
rota virus and noro virus (MNV): 30 sec
adenovirus: 1 mn
polyomavirus (SV 40): 5 mn

* according to recommendation of Robert Koch Institute Berlin / Federal Health
Gazette 01/2004

Warnings / storage

- Inflammable (flash point acc. to DIN 51 755: 26 °C).
- Keep this medical device in a cool – but frost free – and dry place, away from ignition sources and out of reach of children.
- Avoid contact of wipe with eyes.
- Do not use on alcohol-sensitive plastics, e.g. acrylic glass.
- Do not use beyond expiry date.

Properties

- Comprehensive range of action: against bacteria (incl. tb), fungi as well as enveloped viruses and rota viruses
- Pre-cut, to tear off individually
- Ready for use
- Well tolerated by the skin, contains no surfactants, no perfumes
- Whereas other disinfecting and cleaning wipes for surfaces are to be used wearing gloves according to BGR 250/TRBA 250, Hospisept® wipes can be used by unprotected hands
- Suitable for emergency bags (in ambulances, rescue helicopters)
- No hazardous waste left after use

Special precautions

Use medical devices safely! Always read the label and product information before use.

Standard packaging / transport units:

6 x container of 100 wipes

6 x refill bags of 100 wipes

15 x Hospisept® wipes, wrapped individually, packed in small cardboard boxes

If you have questions, ring us up at the following Tel.-no – we shall contact you back immediately!

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

0415-14

Translated by Lysoform Dr. Hans Rosemann GmbH, Berlin - Germany