

*Instructions for use - Please read carefully!*

## **Lydos F – Wall Dispenser**

For 500 ml - bottles

### **Dispenser for liquid soaps and hand disinfectants based on alcohol**

#### **General advice:**

It is recommended to use the dispenser with those products only that are compatible with the material the dispenser is made off.

The dispenser should be occasionally cleaned with warm water and disinfected with disinfectants based on alcohol.

Rests of products do not dry and stick together if the container of the preparation is always filled. In case of blockage please clean with hot water and brush only. Do not unscrew the valve or open the pump!

- 1) Dispenser for liquid hand soaps and hand disinfectants based on alcohol.
- 2) The housing is fixed to the wall plate by clips. Take the housing from the wall plate by pulling forward.
- 3) Installation: in a suitable height with the 4 screws and wall plugs that are enclosed.
- 4) Place the bottle in position and pump several times.
- 5) If necessary: Adjust the dosing limit. Therefore, turn the dosing device made of plastic in the desired position.
- 6) Fix the housing with the clip onto the wall plate.
- 7) Hygienic elbow or forearm-tripping.

#### **Packaging**

Lydos F wall dispenser

**Lysoform Dr. Hans Rosemann GmbH**  
**Kaiser-Wilhelm-Str. 133, D – 12247 Berlin**  
**Tel.: +49-30-77992-0**  
**Telefax: +49- 30-77992-219**