



SURFACE  
DISINFECTION

## Lysoformin® rapid wipes

Impregnated wipes for quick wiping disinfection and cleaning of medical devices and (small) surfaces in patient environments in the medical and veterinarian industry.



CE0482

# Product Information

Please read carefully!

## Lysoformin® rapid wipes

For professional use.

BAuA: Reg-Nr. N-73365

### INGREDIENTS

100 **Lysoformin® rapid wipes** contain:

1.5 l **Lysoformin® rapid** (1%) solution

Active ingredients per wipe:

8.85 mg Didecyldimethylammonium chloride

11.1 mg Bis-(3-aminopropyl)dodecylamine

### PRODUCT FEATURES

- 100 wipes (30 x 30 cm) per roll, a single wipe is sufficient for around 1m<sup>2</sup>.
- Efficient application, wipes can be torn away individually.
- Highly absorbent and lint-free.
- Standing bag with carrying handle for easy use and transportation.
- EN 16615 (4-field-test)
- VAH-listed.

### Disinfection and cleaning of surfaces

Prevention of nosocomial infections	5 min contact time
Virucidal activity against enveloped viruses	10 min contact time

### SCOPE OF APPLICATION

Quick wiping disinfection and cleaning of medical devices and (small) surfaces in the patient environment in the medical and veterinary industries.

### APPLICATION:

Ensure the surface to be disinfected remains completely moistened and allow to operate for the entire duration of the application.

On the round label, the date of the first opening is noted by the user. The impregnated wipes can be used for up to 28 days after opening.



1. Fill out the round label, peel it off and stick it to the lid.



2. Wear protective gloves.



3. Carefully cut out the film on the inside of the black screw cap with a sharp, pointed knife and remove it completely.



4. Pull the first cloth through the opening of the extraction cap and discard.



5. Close the lid of the extraction cap and screw it onto the black screw cap. Always close the lid tightly **immediately** after removing a wipe.

### Reprocessing of the extraction cap in the event of reuse.

Inadequate hygiene can lead to germ colonization of the dispenser system. Because of this, the used extraction cap must be cleaned before every pouch change.

The cap is cleaned by unscrewing the cap, opening it, and rinsing thoroughly under running water. Pay particular attention to the thread of the screw cap. Afterwards, dry the extraction cap with disposable paper towels and store in a dry and protected place until reuse. Alternatively, the extraction cap can also be prepared by machine (max. 60°C). Before screwing the extraction cap into a new bag the disinfection of the catch with an alcoholic surface disinfectant is recommended.

### WARNING AND STORAGE NOTICES

- Use **Lysoformin® rapid wipes** carefully. Always read the label and product information before use.
- Store **Lysoformin® rapid wipes** upright in a closed original container, in a dry, cool but frost-free environment out of reach of children.
- Wear suitable protective gloves when handling the product.

### SHIPPING UNIT/CONTAINER

4 Standing pouches with 4 rolls of 100 wipes  
1 extraction cap  
1 mini-box cutter

### Art. no.

9235  
3405

**LYSOFORM**  
Seit ▪ Since ▪ Depuis 1900

Lysoform Dr. Hans Rosemann GmbH  
Kaiser-Wilhelm-Str. 133, D-12247 Berlin  
Telefon +49 30 77992-0  
www.lysoform.com, e-mail: kontakt@lysoform.de